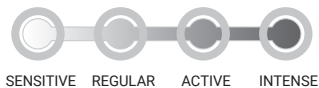
BellaBambi® maxi

Original instructions and translations (English,
French, Spanish) of original instructions

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Caution

This device is not intended to be used by persons (including children) with limited physical, sensory or mental capacities or by persons lacking suitable experience and/or knowledge, unless they are supervised by a responsible person for their protection or have been instructed on the use of the device. Children must be supervised to ensure that they do not play with the device. This device can be used by children aged 8 years or more and by persons with limited physical, sensory or mental capacities or persons lacking suitable experience or knowledge, provided that they are supervised or have received instruction on the safe use of the device and understand the hazards.

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Use and operating principle

Squeeze together, place on your skin and release your grip. You can move the suction cup over your skin or leave it in place (see the example exercises on the reverse).

If you find the application unpleasant, squeeze the BellaBambi® less before placing it on your skin or use a softer intensity. Always pay attention to your sense of wellbeing.

CAUTION:

To avoid skin marks, do not use BellaBambi® longer than 20 seconds in a static location. Never use it in a static location on your face.

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Do not apply on:

Nipples, sexual organs, anus, eyes, eyelids, lips, mouth, lymph nodes, armpits, or the inguinal region.

Consult your physician before use if any of the following conditions applies:

Pregnancy, acute injuries, open wounds, inflammations, varicose veins, disposition to haematoma and ulcers caused by diabetes or vascular diseases, irritable skin disorders such as psoriasis or urticaria (hives), blood-thinning medication therapies, tumours, complex regional pain syndrome (CRPS), as well as any unclear or acute medical conditions.

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Cleaning and storage

BellaBambi® can be cleaned very easily in the dishwasher or by hand with mild soap suds. After drying, treat with a suitable disinfectant for silicones (without phenol derivatives or alkyl amines (glycoproteins)). Please note: If your BellaBambi® is very dirty, you can boil it like a pacifier. Use a clean pot, cover your BellaBambi® completely with water, and boil it for at least five minutes.

Do not allow children to clean the suction cup without supervision.

Suitable cleaning agents and disinfectants can be found on www.bellabambi.com

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Recommendations

An oil to assist lubricity is helpful for some applications. BellaBambi® can also be used with hyaluronan liquid, cellulite creams or body lotions.

Warranty

We provide a 2 year manufacturer's warranty on all suction cups. Please note that this warranty is subject to the exclusive use of related products and cleaning agents recommended by the manufacturer. The warranty conditions are available at www.bellabambi.com/warranty

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